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REF. 1008789

INSTRUCTION FOR USE

AC50 Barrier mask according to AFNOR SPEC S76-001

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WARNINGS

The **AC50 barrier mask** is intended for use by healthy people without clinical symptoms of viral infection. If you are sick, seek the advice of your doctor. It does not replace hygienic protective actions and rules of physical distance. It constitutes a protective barrier against possible viral penetration into the mouth and nose area of its user or a person being nearby. It also aims to protect this area against all contact with the hands. This equipment is neither a medical device within the meaning of Regulation EU/2017/745 (surgical masks), nor a personal protective equipment within the meaning of Regulation EU/2016/425 (filtering masks of type FFP2). It does not replace the Personal Protection Equipment (PPE) compulsory in some occupations. In addition, it is reminded that the use of masks in the professional environment must be part of the implementation by the employer of the measures necessary to ensure the safety and protect the health of workers. In the same way, the employer must ensure that the effective breathability of the mask is compatible with the specific conditions of the workplace and in particular to the effort undertaken. It is not intended for medical staff. The **AC50 mask** has maximum effectiveness if it is worn in direct contact with bare skin, avoiding contact with the hair. Always check that the mask is properly adjusted and covers your mouth and nose. The maximum duration of use of the **AC50 mask** is 4 hours. It must be washed or discarded afterwards, in accordance with the instructions given in this instruction for use. The complete mask must imperatively be checked after each washing and drying: absence of hole and mechanical degradation of the material (abrasion, scratches, etc.), appearance and function of the elastics, integrity of the seams. **To protect your health and that of others, it is very important to follow these instructions for use. MATISEC declines any responsibility in the event of use of mask AC50 apart from the recommendations and uses described.**

GENERAL DESCRIPTION AND LEVELS OF PROTECTION

The **AC50 barrier mask** is designed according to the AFNOR SPEC S76-001 guide and meets the requirements of the **French interministerial note dated March 29th, 2020**, for use as :

- Mask for non-sanitary use of **Category 1 - UNS 1**: individual mask for use by professionals in contact with the public (people assigned to positions or missions involving regular contact with the public). Test according to annex 2 of the DGA protocol: protection efficiency against particles of 3 µm > 90%, new and after washing.
- Mask to **protect the wearer** against particles smaller than 3 µm. Test according to annex 1 of the DGA protocol: protection efficiency for particles of 1 µm > 30%, new and after washing. *As a reminder, the filtration efficiency of a FFP2 respiratory protection mask is at least 94%.*

For these mask categories, the following requirements apply :

- Air permeability for a vacuum of 100 Pa > 96 Lm².s⁻¹, new and after washing.
- Respiratory resistance in sinusoidal flow 30 l / min < 2.4 mbar on inspiration and < 3 mbar on expiration, new and after washing.

CHARACTERISTICS

- One size - Adult
- "Pleated" mask made of laminated 2-layer PES 285 g/m² fabric.
- 2 elastic bands for tightening and adjustment at the back of the head.
- Side elastic to seal the mask on the face.
- Adjustable nose clamp.
- The material of the mask does not contain any substance at levels known to be dangerous for the user.

CLEANING

The efficiency of the material have been validated new and after 50 cycles of washing and drying. The **AC50 mask** can be washed up to 50 times according to the following procedure :

- Machine wash with conventional detergent, without additives, for at least 30 min at 60° C.
- Mechanical drying at a maximum temperature of 60° C.
- Conventional drying in the open air / steam ironing.

The complete mask must imperatively be checked after each cycle (washing and drying): absence of hole and mechanical degradation of the material (abrasion, scratches, etc.), appearance and function of the elastic, integrity of seams.

It is recommended to :

- Carry out a washing cycle before the first use.
- Use a laundry net.

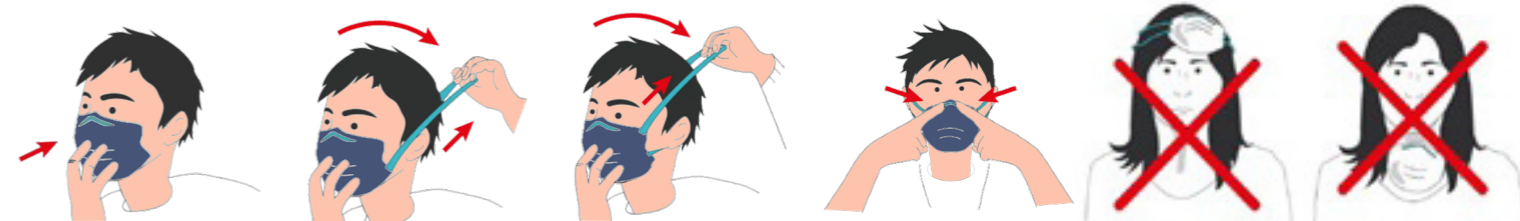
Lot / Batch n° : ww/yy

PACKAGING & STORAGE

Transparent plastic bag or cardboard box of 20 masks. Storage between + 5 ° C and + 45 ° C in a dry and ventilated room, away from dust and light.

USE

Wash your hands with soap and water or a alcoholic solution.
Inspect the mask and verify that there are no holes, cracks or damage.
Place the mask on the face. The pictogram must be on the outside, down right.
Hold the mask from the outside and pass the elastic bands to the back of the head.
Pinch the nose clamp and adjust until there is no air passage at eye level.
Check that the mask is correctly fitted: fitting under the chin, covering the mouth and nose. If necessary, the length of the elastics can be reduced with a knot.
Once adjusted, do not touch the mask with your hands. Before the mask is touched, wash your hands with soap and water or an alcoholic solution.
Remove the mask by grabbing the elastics at the back of the head, without touching the front part of the mask.
The mask should be washed after 4 hours of use.
During the same period of 4 hours, the mask can only be used several times if it is removed according to the instructions given in this instruction, temporarily stored or hung to provide the smallest possible contact and put back according to this instruction.
Change your mask if it is deformed, damaged, soiled, wet or if it becomes difficult to breathe.
The mask should be discarded in a bin with a plastic bag. Double packaging is recommended.
Between active use of the mask, it must not be put on the forehead or on the chin.
Never modify the mask in any way.

This document is the property of MATISEC and shall not be used, copied or communicated without the company's consent.

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